

CASE STUDY



Dr Annabelle Trainor BSc, DVM

CROWN EQUINE VETERINARY SERVICES – MELBOURNE

EQUINE - Leg wound trauma

TRAUMA 7th October 2019
Horizontal Laceration

2 y old TB Colt
Kicked into a wall
1 week AB + Bute

29th Oct 2020
X-Ray rules out
sequestration

Non healing
3 weeks



K-LASER THERAPY

5 Sessions over 1 month

Settings for chronic wounds

4 wavelengths - 660 -800 -905 -970nm

Average Power 3 watts

11 phases with

Continuous wave and

Pulsing 10 to 10.000Hz



1rst K-Laser
Session

Healing

29th Oct 2019

19th Nov 2019

29th Nov 2019

3rd Dec 2019

